

Silicone Gastrostomy Tube

Instructions for Use

Doc. No.:CE/TD-GT-22-01

Ver.:A.00

Date of Release:2025-07-28


【Warning】

1. Please read this instruction before use. Use strictly according to the specifications in the instruction.
2. This product is sterilized with ethylene oxide, with a shelf life of 3 years.
3. Do not use after expiration. The production date and expiration date are indicated on the product packaging.
4. Carefully check the packaging for integrity before use; do not use if the primary packaging is damaged.
5. This product is for single use only; reuse is strictly prohibited. Attempts to reprocess, re-sterilize, and/or reuse may result in product failure and/or transmission of disease.
6. Be sure to use a Gastrostomy tube that corresponds to the size of the fistula (generally the same as before exchange gastrostomy tube); an oversized Gastrostomy tube may cause inability to retain or damage the fistula.
7. During placement, be careful not to apply excessive pressure between the abdominal wall and gastric wall, and appropriately set the position of the Fixed ring (Y type, E type, VED series).
8. When removing the Gastrostomy tube, if it becomes adhered to the fistula, do not forcibly pull it out; it must be removed using an endoscope. Otherwise, it may cause damage to the mucosal tissue or rupture of the Gastrostomy tube.
9. Be sure to confirm that the Gastrostomy tube is retained in the stomach before starting nutritional input. If nutritional input is performed while the Gastrostomy tube has fallen into the abdominal cavity, it may lead to severe complications such as peritonitis, and may even result in patient death. If nutritional input is inadvertently delivered into the abdominal cavity, contact a doctor immediately and take appropriate measures.
10. During indwelling, the Balloon may experience natural leakage; the Balloon may also shrink, so it is important to regularly monitor the Balloon's inflation status.
11. Any serious incidents related to this product should be reported to the competent authorities of the EU member state where the manufacturer, user and/or patient are located.

[PRODUCT NAME] Silicone Gastrostomy Tube

[INTENDED USE] This product is suitable for the replacement of existing devices using established stoma channels. It can directly deliver enteral nutrition and medications to the stomach and/or perform gastric decompression.


[PRODUCT COMPONENTS] The Silicone Gastrostomy Tube (hereinafter referred to as Gastrostomy tube) consists of the Tube body, Balloon, Feeding joint, Fixed ring (N type without Fixed ring), Filling joint, and other components. Product specifications are classified based on Tube body diameter, Feeding joint type, Fixed ring type, Tube body length. The basic structural components are shown in Figures 1 to 6, and the specifications are listed in Appendix A.

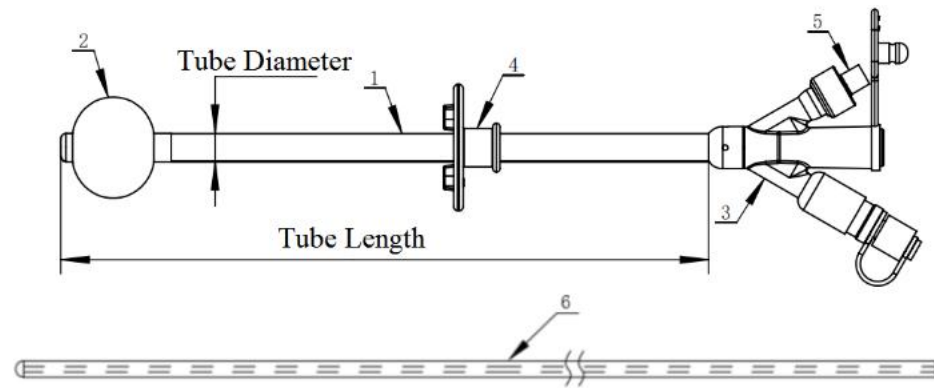

1 Tube body 2 Balloon 3 Feeding joint 4 Fixed ring 5 Filling joint 6 Guide bar

(Y-type and YC-type do not include 6)


Figure 1: Schematic composition diagram of GT Series Y-type Silicone Gastrostomy Tube

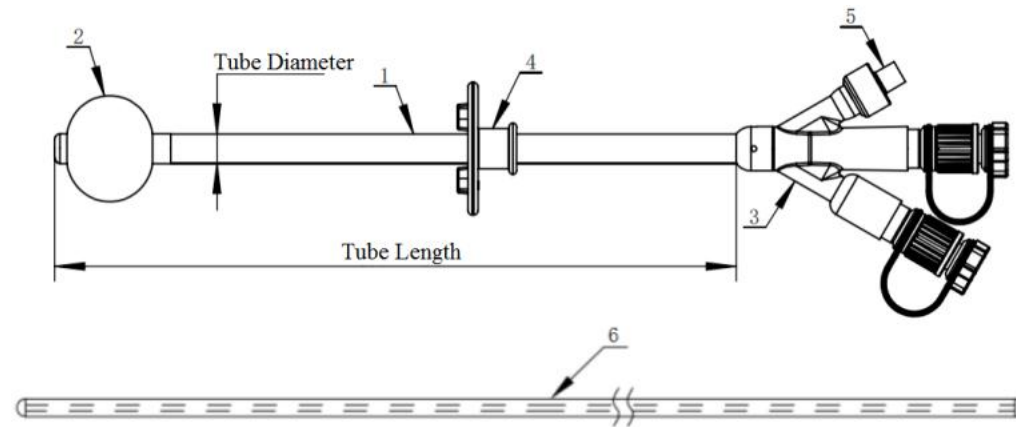

1 Tube body 2 Balloon 3 Feeding joint 4 Fixed ring 5 Filling joint 6 Guide bar



(E-type and EC-type do not include 6)


Figure 2: Schematic composition diagram of GT Series E-type Silicone Gastrostomy Tube

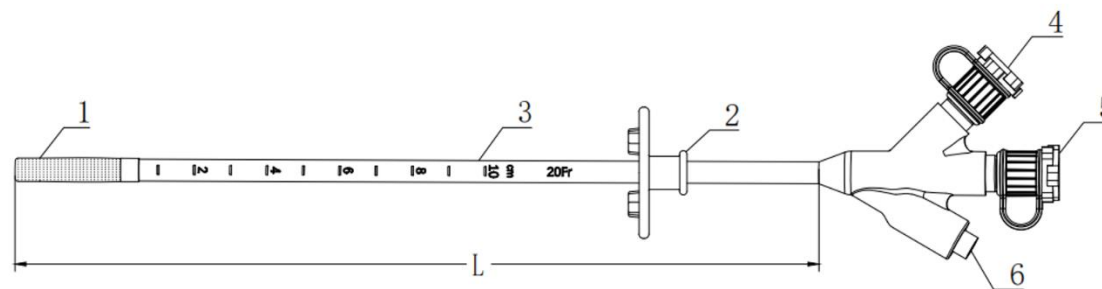

1 Balloon 2 Fixed ring 3 Tube body 4 Feeding joint 5 Feeding joint 6 Filling joint

Figure 3: Schematic composition diagram of VED Series E-type Silicone Gastrostomy Tube

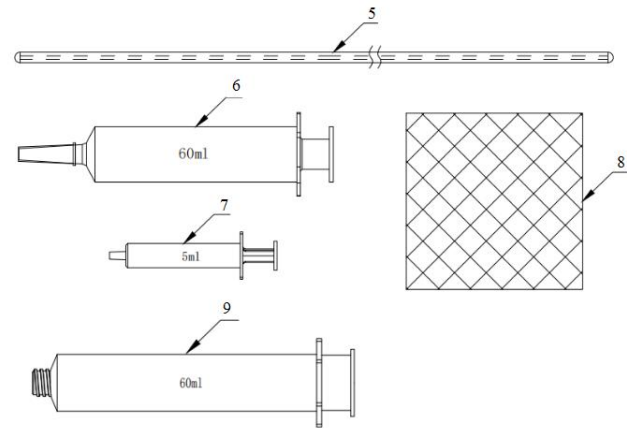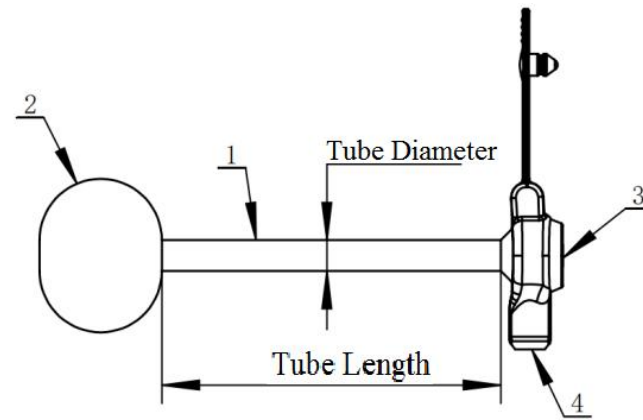

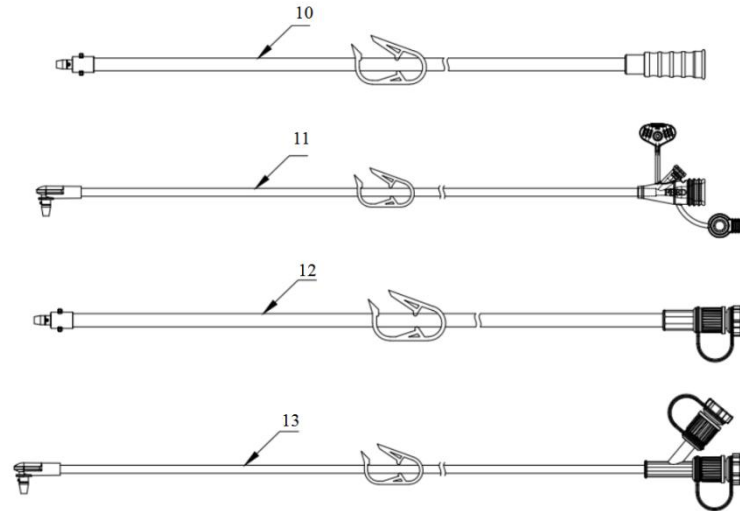

1 Tube body 2 Balloon 3 Feeding joint 4 Filling joint 5 Guide bar 6 Feeder 7 Injector 8 Gauze sponges 9 E-type Feeder 10 Straight feeding tube

11 Bent feeding tube 12 E-type Straight feeding tube 13 E-type Bent feeding tube

(N-type include 1, 2, 3, 4; NL-type include 1, 2, 3, 4, 5, 6, 7, 8, 10, 11; NLE-type include 1, 2, 3, 4, 5, 7, 8, 9, 12, 13)

Figure 4: Schematic composition diagram of N-type Silicone Gastrostomy Tube

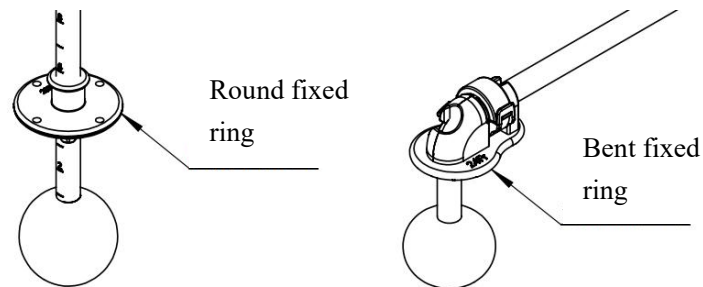


Figure 5 Schematic diagram of Fixed ring types

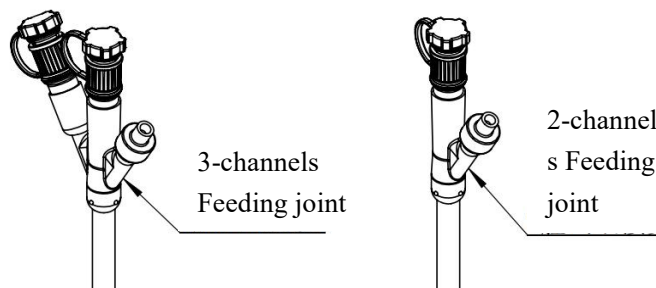


Figure 5 Schematic diagram of Feeding joint channel types

[Specification and model]

1. Specification and model table

The Silicone Gastrostomy Tube is classified into Y type, E type, VED series, and N type, as detailed in Appendix A.

Note: The specific specifications and models of this product are indicated on the primary packaging label.

[PRODUCT PERFORMANCE]

1. The flow rate of the Gastrostomy tube should not be less than 9 mL/min.
2. The connection between the Feeding joint and the Feeder or Enteral nutrition catheter should withstand a linear tensile force of 15N without breaking or separating.
3. For the Balloon with a rated volume of 5ml, the burst volume should be $\geq 10\text{ml}$; For a Balloon with a rated volume of 10ml, the burst volume should be $\geq 20\text{ml}$; For a Balloon with a rated volume of 20ml, the burst volume should be $\geq 40\text{ml}$.
4. This product is for single use, sterilized with ethylene oxide, provided sterile, and has a shelf life of 3 years.

[INDICATIONS]

Patients with transoral feeding disorders caused by various reasons, but with normal gastrointestinal function, and who need long-term tube feeding for nutrition and/or medications, as well as gastrointestinal decompression. The specific indications are as follows:

- Patients with swallowing disorders due to central nervous system diseases (e.g., stroke, traumatic brain injury, vegetable, etc.);
- Patients undergoing radiotherapy or surgery for head and neck tumors (nasopharyngeal, oral);
- Esophageal perforation, esophageal fistula, or extensive scarring of the esophagus;
- Insufficient intake (e.g., burns, AIDS, anorexia, bone marrow transplant recipients);
- Extrahepatic fistula or biliary drainage;
- Severe pancreatitis, pancreatic cysts, or gastric emptying disorders (jejunostomy feeding tube);
- Persistent and refractory vomiting due to various causes (e.g., chemotherapy for tumors).

[CONTRAINDICATIONS]

Contraindications for placement of a gastrostomy feeding tube include, but are not limited to ascites, colonic interposition, portal hypertension, peritonitis and morbid obesity.

[POTENTIAL COMPLICATIONS]

Potential complications associated with the indwelling and use of The Silicone Gastrostomy Tube include, but are not limited to:

- Skin breakdown
- Infection
- Hypergranulation tissue
- Gastric or duodenal ulcers
- Intraperitoneal leakage

- Pressure necrosis

[CLINICAL BENEFITS]

Provide patients with effective enteral nutrition/medication support to improve patients' quality of life/nutritional status/length of stay.

[TARGET POPULATION]

Patients with transoral feeding disorders caused by various reasons, but with normal gastrointestinal function, and who need long-term tube feeding for nutrition and/or medications, as well as gastrointestinal decompression

[INTENDED USERS]

The operators of this device must be physicians or medical personnel under the supervision of a physician, and they must have received adequate clinical technical training.

[PRECAUTIONS]

1. Reuse is prohibited.
2. Do not use this product if a fistula has not actually formed, or if there is damage or abnormalities in the fistula.
3. Use sterilized distilled water for Balloon inflation; using saline, contrast agents, etc., may cause the Balloon to become non-retractable due to coagulation of solution composition.
4. Do not perform Balloon inflation or nutritional input when the Gastrostomy tube is in a horizontal fixed position.
5. During placement and use of the Gastrostomy tube, care must be taken to avoid kinking or damaging any components of the tube.
6. Do not use any method to modify the Gastrostomy tube or connectors.

7. Strictly follow the instructions for use in each accompanying device. The manufacturer's instructions do not replace medical advice.
8. Before using this device, it is essential to fully understand its mechanical principles, clinical applications, and the risks associated with the placement and/or removal of the Gastrostomy tube. Placement and/or removal of the Gastrostomy tube must be performed by fully trained physicians or under their supervision.
9. When placing a Gastrostomy tube in obese patients, all anatomical structures must be identified prior to placement.
10. Adhere to all regulatory requirements for endoscopic procedures when placing the Gastrostomy tube, including the removal of dentures.
11. It is recommended to replace the Gastrostomy tube within 30 days.

[INSTRUCTIONS FOR USE]

1.Preparation of the Gastrostomy tube: Select the appropriate gastrostomy tube (choose the suitable outer diameter for Y type, E type, and VED series; for N type, select the suitable outer diameter and Tube body length). Remove it from the packaging and check for damage. Use the injector to inject sterile water or distilled water into the Balloon through the Filling joint, according to the Balloon's labeled capacity. After removing the injector, gently squeeze the Balloon to check for leaks and verify its integrity. Visually inspect the Balloon for symmetry. Rolling the Balloon gently between your fingers can help achieve symmetry. Reinsert the injector and withdraw all water from the Balloon.

[Warning] For infants and children, the insertion site should be at a higher position on a larger curvature of the stomach to prevent obstruction of the pylorus during Balloon inflation.

- 2.Apply a water-soluble lubricant to the tip of the Gastrostomy tube; do not use mineral oil or vaseline.
- 3.Position the patient supine, clean and disinfect the stoma site. Hold the proximal end of the Gastrostomy tube and slowly rotate the tube body

while gently advancing it 1-2 cm into the stomach, allowing the Gastrostomy tube to separate from the stoma channels.

[Warning] If the Gastrostomy tube does not rotate easily, do not use this pulling method to remove the tube.

Apply lubricant to the lumen of the Gastrostomy tube, insert the guide bar, cover the stoma site with a piece of medical gauze sponges, place one hand on the base of the Gastrostomy tube, and use the other hand to pull oppositely the Gastrostomy tube out of the fistula, being careful not to pull out the guide bar at the same time.

4. Apply lubricant at the fistula site and slowly insert the front end of Silicone Gastrostomy Tube into the fistula along the guide bar, going slightly deeper than the original Gastrostomy tube to ensure the Balloon is fully inside the stomach. Use the injector to slowly inject sterile water or distilled water into the Balloon through the Filling joint, according to the Balloon's labeled capacity.

5. Warning: Sterile water or distilled water should be injected into the Balloon according to its labeled capacity; do not inject air into the Balloon.

6. Slowly lift the Gastrostomy tube until the Balloon contacts the stomach wall, feeling slight resistance, while simultaneously pushing down the external Fixed ring (N type has no Fixed ring), but do not tighten excessively; the patient should not feel pain. If the patient feels pain, it indicates that the Gastrostomy tube may not be properly positioned in the stomach; confirmation of correct placement can be achieved by sucking out gastric contents. If gastric contents can be aspirated, it indicates the Gastrostomy tube is effectively positioned. If gastric contents cannot be sucked out, it means the Gastrostomy tube is incorrectly placed and needs to be repositioned. Feeding should only begin after confirming appropriate patency and placement, and following the physician's instructions.

Note: For the Bent fixed ring, insert the Gastrostomy tube into the guiding hole of the bent fixed plate, and when traction is established between the Bent fixed ring and the abdominal wall, secure it using the attached clamp.

[Warning] The most serious complication of replacing the Gastrostomy tube is placing the end of the tube in subcutaneous tissue or the abdominal cavity instead of the stomach. Providing nutritional support with a Gastrostomy tube not placed in the stomach can be life-threatening; an endoscope should be used if necessary to confirm that it is properly placed in the stomach.

7. Check for signs of gastric fluid leakage around the stoma, and verify the position of the Gastrostomy tube and the placement of the Fixed ring (N type has no Fixed ring). Feeding should only commence after confirming appropriate patency and placement, and following the physician's instructions.

【POSTOPERATIVE STEPS】

1. After the placement of the Gastrostomy tube, feeding ratios and volumes should be based on the physician's prescription.
2. Medication Administration, use liquid medication. Solid medication need to consult the pharmacist to determine if it is safe to crush solid medication and mix with water. If safe, pulverize the solid medication into a fine powder form and dissolve the powder in water before administering through the feeding tube. Never crush enteric coated medication or mix medication with formula. Using a catheter tip syringe flush the tube with the prescribed amount of water.
3. The physician should establish an antibiotic regimen to prevent infection and assess the patient for any signs of infection, such as redness, irritation, edema, swelling, tenderness, fever, rash, purulence, or gastrointestinal drainage. Assess for signs of patient's pressure necrosis, skin breakdown, or granulation tissue proliferation.
4. Keep the skin around the wound clean, dry, and free from leakage. The wound and surrounding skin should be checked before each feeding, and the area should be cleaned at least once daily with a mild soap solution, ensuring thorough cleaning underneath the Fixed ring. Be sure to rinse thoroughly and dry the skin.
5. Assess the Gastrostomy tube for any abnormalities, such as blockages or unusual discoloration.
6. Clean the Feeding joint using mild soap and water, being careful not to pull or manipulate the Gastrostomy tube excessively. Rinse thoroughly

and let dry.

7. Rotate the Gastrostomy tube gently, turning it 360 degrees plus a quarter turn daily.
8. Flush the Gastrostomy tube with water during feeding interruptions or flush the ostomy tube with water at least every 8 hours
9. For a blocked Gastrostomy tube, ensure it is not kinked or pinched. If the blockage is visible above the skin surface, gently massage or press the tube with your fingers to dislodge it. Next, insert a Injector filled with warm water into the Feeding joint and gently inject it into the Gastrostomy tube to remove the blockage. If the blockage persists, consult a physician.
10. After the wound has healed, the Gastrostomy tube no longer requires a dressing. Dressings may cause skin irritation and dehydration. Keep the skin around the stoma clean, dry, and free from drainage.
11. Balloon maintenance: Check the water volume in the Balloon weekly. Insert the injector into the Filling adapter and, while keeping the tube in position, withdraw the liquid. Compare the amount of water in the injector with the recommended volume. If the volume is less than the suggested or specified amount, refill the Balloon with the initial amount of water removed, then add the required amount of water to reach the recommended or specified volume. Wait 10-20 minutes and repeat this step. If the Balloon's water volume is still less than the recommended or specified amount, it indicates a leak, and a new Balloon Gastrostomy tube should be replaced.
12. Balloon lifespan: The exact lifespan of the Balloon is unpredictable. Silicone balloons typically last for 30 days, but the lifespan can vary based on individual factors. These factors may include medications, the amount of water used to fill the Balloon, gastric pH, and care of the Balloon tube.

[Removing the Gastrostomy Tube]

1. Prepare all equipment and supplies, use sterile techniques to clean your hands, and put on powder-free cleaning gloves.
2. Rotate the tube 360 degrees to ensure it moves freely and easily.

[Warning] If the tube body does not rotate easily in the pathway, do not attempt to use this pulling method to remove the Gastrostomy Tube.

3. Firmly insert the injector into the Balloon Filling adapter and withdraw all fluids.
4. Hold the Gastrostomy tube near the stoma and place a finger from the other hand at the base of the tube body to apply opposing force.
5. Loosely cover the stoma area with a towel or surgical drape.
6. Hold the Gastrostomy tube straight and pull with steady traction until the internal curved portion emerges from the abdominal wall.

[Warning] The Gastrostomy Tube must be pulled straight out from the stoma channel. If a replacement tube is needed, it must be inserted immediately after the removal of the Gastrostomy Tube. Movement of the external Gastrostomy Tube may cause very minor trauma or bleeding that could require medical attention.

[WASTE DISPOSAL]

Silicone Gastrostomy Tube should be disposed of by the using healthcare organisation in accordance with the disposal procedures for disposable medical consumables.

[DATE OF MANUFACTURE]

See the label on product primary package.

[STERILIZATION METHOD]

Ethylene oxide sterilization.

[EXPIRATION DATE]

3 years

[STORAGE AND TRANSPORTATION CONDITIONS]

The product should be protected from heavy pressure, direct sunlight, and rain during transportation. It should be stored indoors in a clean, well-ventilated environment, avoiding sunlight and corrosive gases.

[SYMBOLS DESCRIPTION]

	Manufacturer		Authorized representative in the European Community
	Date of manufacture		Expiration date
	Unique device identifier		Medical device
	Specification and model		Lot number
	Do not use if primary package is damaged		Do not re-use
	Consult instructions for use		Caution
	Sterilized using ethylene oxide, single sterile barrier system with protective packaging outside		Stacking limit by number
	Keep away from sunlight		Keep away from rain
	Package quantity		CE marking

[CONTACT INFORMATION]



Beijing ZKSK Technology Co., Ltd.

Building 9, 6 & No.6 Yuan Hengye, North 7th street, Yongle Economic Development Zone, Tongzhou District, 101105 Beijing,
PEOPLE'S REPUBLIC OF CHINA

Postcode:101105

Tel.: +86-10-80525660

Website: <http://www.bjzksk.com>

Appendix A

Table 1 Specification and model table of Silicone Gastrostomy Tube

No.	GT Series Y-type joint (3-channels)		GT Series E-type (ENFit) joint (3-channels)		Gastrostomy Tube Diameter		Fixed Ring Type	Balloon Rated Volume±2 mL	Tube Body Length
					French Specificati on	Millimeter Size			
1	GT12F-Y	GT12F-YL	GT12F-E	GT12F-EL	12Fr	4.0	Round	5	185
2	GT14F-Y	GT14F-YL	GT14F-E	GT14F-EL	14Fr	4.7	Round	5	
3	GT16F-Y	GT16F-YL	GT16F-E	GT16F-EL	16Fr	5.3	Round	20	
4	GT18F-Y	GT18F-YL	GT18F-E	GT18F-EL	18Fr	6.0	Round	20	
5	GT20F-Y	GT20F-YL	GT20F-E	GT20F-EL	20Fr	6.7	Round	20	
6	GT22F-Y	GT22F-YL	GT22F-E	GT22F-EL	22Fr	7.3	Round	20	
7	GT24F-Y	GT24F-YL	GT24F-E	GT24F-EL	24Fr	8.0	Round	20	
8	GT26F-Y	GT26F-YL	GT26F-E	GT26F-EL	26Fr	8.7	Round	20	
9	GT12F-YC	GT12F-YCL	GT12F-EC	GT12F-ECL	12Fr	4.0	Bent	5	185
10	GT14F-YC	GT14F-YCL	GT14F-EC	GT14F-ECL	14Fr	4.7	Bent	5	
11	GT16F-YC	GT16F-YCL	GT16F-EC	GT16F-ECL	16Fr	5.3	Bent	20	
12	GT18F-YC	GT18F-YCL	GT18F-EC	GT18F-ECL	18Fr	6.0	Bent	20	
13	GT20F-YC	GT20F-YCL	GT20F-EC	GT20F-ECL	20Fr	6.7	Bent	20	
14	GT22F-YC	GT22F-YCL	GT22F-EC	GT22F-ECL	22Fr	7.3	Bent	20	
15	GT24F-YC	GT24F-YCL	GT24F-EC	GT24F-ECL	24Fr	8.0	Bent	20	
16	GT26F-YC	GT26F-YCL	GT26F-EC	GT26F-ECL	26Fr	8.7	Bent	20	
No.	GT Series Y-type joint (2-channels)		GT Series E-type (ENFit) joint (2-channels)		Gastrostomy Tube Diameter		Fixed Ring Type	Balloon Rated Volumem L	Tube Body Length
					French Specificati on	Millimeter Size			

1	GT12F-Y2	GT12F-YL2	GT12F-E2	GT12F-EL2	12Fr	4.0	Round	5	185
2	GT14F-Y2	GT14F-YL2	GT14F-E2	GT14F-EL2	14Fr	4.7	Round	5	
3	GT16F-Y2	GT16F-YL2	GT16F-E2	GT16F-EL2	16Fr	5.3	Round	20	
4	GT18F-Y2	GT18F-YL2	GT18F-E2	GT18F-EL2	18Fr	6.0	Round	20	
5	GT20F-Y2	GT20F-YL2	GT20F-E2	GT20F-EL2	20Fr	6.7	Round	20	
6	GT22F-Y2	GT22F-YL2	GT22F-E2	GT22F-EL2	22Fr	7.3	Round	20	
7	GT24F-Y2	GT24F-YL2	GT24F-E2	GT24F-EL2	24Fr	8.0	Round	20	
8	GT26F-Y2	GT26F-YL2	GT26F-E2	GT26F-EL2	26Fr	8.7	Round	20	
9	GT12F-YC2	GT12F-YCL2	GT12F-EC2	GT12F-ECL2	12Fr	4.0	Bent	5	185
10	GT14F-YC2	GT14F-YCL2	GT14F-EC2	GT14F-ECL2	14Fr	4.7	Bent	5	
11	GT16F-YC2	GT16F-YCL2	GT16F-EC2	GT16F-ECL2	16Fr	5.3	Bent	20	
12	GT18F-YC2	GT18F-YCL2	GT18F-EC2	GT18F-ECL2	18Fr	6.0	Bent	20	
13	GT20F-YC2	GT20F-YCL2	GT20F-EC2	GT20F-ECL2	20Fr	6.7	Bent	20	
14	GT22F-YC2	GT22F-YCL2	GT22F-EC2	GT22F-ECL2	22Fr	7.3	Bent	20	
15	GT24F-YC2	GT24F-YCL2	GT24F-EC2	GT24F-ECL2	24Fr	8.0	Bent	20	
16	GT26F-YC2	GT26F-YCL2	GT26F-EC2	GT26F-ECL2	26Fr	8.7	Bent	20	
No.	-	-	VED Series E-type (ENFit) joint (3-channels)		Gastrostomy Tube Diameter		Fixed Ring Type	Balloon Rated Volumem L	Tube Body Length
	-	-			French Specificati on	Millimeter Size			
1	-	-	VED-212EO	-	12Fr	4.0	Round	5	220
2	-	-	VED-214EO	-	14Fr	4.7	Round	5	
3	-	-	VED-216EO	-	16Fr	5.3	Round	20	
4	-	-	VED-218EO	-	18Fr	6.0	Round	20	
5	-	-	VED-220EO	-	20Fr	6.7	Round	20	
6	-	-	VED-222EO	-	22Fr	7.3	Round	20	
7	-	-	VED-224EO	-	24Fr	8.0	Round	20	
8	-	-	VED-226EO	-	26Fr	8.7	Round	20	

No.	GT Series N-type joint			Gastrostomy Tube Diameter		Fixed Ring Type	Balloon Rated Volumem L	Tube Body Length
				French Specification	Millimeter Size			
1	GT12F-N8	GT12F-N8L	GT12F-N8LE	12Fr	4.0	-	5	8
2	GT14F-N8	GT14F-N8L	GT14F-N8LE	14Fr	4.7	-	5	
3	GT16F-N8	GT16F-N8L	GT16F-N8LE	16Fr	5.3	-	10	
4	GT18F-N8	GT18F-N8L	GT18F-N8LE	18Fr	6.0	-	10	
5	GT20F-N8	GT20F-N8L	GT20F-N8LE	20Fr	6.7	-	10	
6	GT22F-N8	GT22F-N8L	GT22F-N8LE	22Fr	7.3	-	10	
7	GT24F-N8	GT24F-N8L	GT24F-N8LE	24Fr	8.0	-	10	
8	GT26F-N8	GT26F-N8L	GT26F-N8LE	26Fr	8.7	-	10	
9	GT12F-N10	GT12F-N10L	GT12F-N10LE	12Fr	4.0	-	5	10
10	GT14F-N10	GT14F-N10L	GT14F-N10LE	14Fr	4.7	-	5	
11	GT16F-N10	GT16F-N10L	GT16F-N10LE	16Fr	5.3	-	10	
12	GT18F-N10	GT18F-N10L	GT18F-N10LE	18Fr	6.0	-	10	
13	GT20F-N10	GT20F-N10L	GT20F-N10LE	20Fr	6.7	-	10	
14	GT22F-N10	GT22F-N10L	GT22F-N10LE	22Fr	7.3	-	10	
15	GT24F-N10	GT24F-N10L	GT24F-N10LE	24Fr	8.0	-	10	
16	GT26F-N10	GT26F-N10L	GT26F-N10LE	26Fr	8.7	-	10	
17	GT12F-N12	GT12F-N12L	GT12F-N12LE	12Fr	4.0	-	5	12
18	GT14F-N12	GT14F-N12L	GT14F-N12LE	14Fr	4.7	-	5	
19	GT16F-N12	GT16F-N12L	GT16F-N12LE	16Fr	5.3	-	10	
20	GT18F-N12	GT18F-N12L	GT18F-N12LE	18Fr	6.0	-	10	
21	GT20F-N12	GT20F-N12L	GT20F-N12LE	20Fr	6.7	-	10	
22	GT22F-N12	GT22F-N12L	GT22F-N12LE	22Fr	7.3	-	10	
23	GT24F-N12	GT24F-N12L	GT24F-N12LE	24Fr	8.0	-	10	
24	GT26F-N12	GT26F-N12L	GT26F-N12LE	26Fr	8.7	-	10	
25	GT12F-N15	GT12F-N15L	GT12F-N15LE	12Fr	4.0	-	5	15

26	GT14F-N15	GT14F-N15L	GT14F-N15LE	14Fr	4.7	-	5	
27	GT16F-N15	GT16F-N15L	GT16F-N15LE	16Fr	5.3	-	10	
28	GT18F-N15	GT18F-N15L	GT18F-N15LE	18Fr	6.0	-	10	
29	GT20F-N15	GT20F-N15L	GT20F-N15LE	20Fr	6.7	-	10	
30	GT22F-N15	GT22F-N15L	GT22F-N15LE	22Fr	7.3	-	10	
31	GT24F-N15	GT24F-N15L	GT24F-N15LE	24Fr	8.0	-	10	
32	GT26F-N15	GT26F-N15L	GT26F-N15LE	26Fr	8.7	-	10	
33	GT12F-N17	GT12F-N17L	GT12F-N17LE	12Fr	4.0	-	5	17
34	GT14F-N17	GT14F-N17L	GT14F-N17LE	14Fr	4.7	-	5	
35	GT16F-N17	GT16F-N17L	GT16F-N17LE	16Fr	5.3	-	10	
36	GT18F-N17	GT18F-N17L	GT18F-N17LE	18Fr	6.0	-	10	
37	GT20F-N17	GT20F-N17L	GT20F-N17LE	20Fr	6.7	-	10	
38	GT22F-N17	GT22F-N17L	GT22F-N17LE	22Fr	7.3	-	10	
39	GT24F-N17	GT24F-N17L	GT24F-N17LE	24Fr	8.0	-	10	
40	GT26F-N17	GT26F-N17L	GT26F-N17LE	26Fr	8.7	-	10	20
41	GT12F-N20	GT12F-N20L	GT12F-N20LE	12Fr	4.0	-	5	
42	GT14F-N20	GT14F-N20L	GT14F-N20LE	14Fr	4.7	-	5	
43	GT16F-N20	GT16F-N20L	GT16F-N20LE	16Fr	5.3	-	10	
44	GT18F-N20	GT18F-N20L	GT18F-N20LE	18Fr	6.0	-	10	
45	GT20F-N20	GT20F-N20L	GT20F-N20LE	20Fr	6.7	-	10	
46	GT22F-N20	GT22F-N20L	GT22F-N20LE	22Fr	7.3	-	10	
47	GT24F-N20	GT24F-N20L	GT24F-N20LE	24Fr	8.0	-	10	
48	GT26F-N20	GT26F-N20L	GT26F-N20LE	26Fr	8.7	-	10	23
49	GT12F-N23	GT12F-N23L	GT12F-N23LE	12Fr	4.0	-	5	
50	GT14F-N23	GT14F-N23L	GT14F-N23LE	14Fr	4.7	-	5	
51	GT16F-N23	GT16F-N23L	GT16F-N23LE	16Fr	5.3	-	10	
52	GT18F-N23	GT18F-N23L	GT18F-N23LE	18Fr	6.0	-	10	
53	GT20F-N23	GT20F-N23L	GT20F-N23LE	20Fr	6.7	-	10	
54	GT22F-N23	GT22F-N23L	GT22F-N23LE	22Fr	7.3	-	10	
55	GT24F-N23	GT24F-N23L	GT24F-N23LE	24Fr	8.0	-	10	

56	GT26F-N23	GT26F-N23L	GT26F-N23LE	26Fr	8.7	-	10	
57	GT12F-N25	GT12F-N25L	GT12F-N25LE	12Fr	4.0	-	5	25
58	GT14F-N25	GT14F-N25L	GT14F-N25LE	14Fr	4.7	-	5	
59	GT16F-N25	GT16F-N25L	GT16F-N25LE	16Fr	5.3	-	10	
60	GT18F-N25	GT18F-N25L	GT18F-N25LE	18Fr	6.0	-	10	
61	GT20F-N25	GT20F-N25L	GT20F-N25LE	20Fr	6.7	-	10	
62	GT22F-N25	GT22F-N25L	GT22F-N25LE	22Fr	7.3	-	10	
63	GT24F-N25	GT24F-N25L	GT24F-N25LE	24Fr	8.0	-	10	
64	GT26F-N25	GT26F-N25L	GT26F-N25LE	26Fr	8.7	-	10	
65	GT12F-N27	GT12F-N27L	GT12F-N27LE	12Fr	4.0	-	5	27
66	GT14F-N27	GT14F-N27L	GT14F-N27LE	14Fr	4.7	-	5	
67	GT16F-N27	GT16F-N27L	GT16F-N27LE	16Fr	5.3	-	10	
68	GT18F-N27	GT18F-N27L	GT18F-N27LE	18Fr	6.0	-	10	
69	GT20F-N27	GT20F-N27L	GT20F-N27LE	20Fr	6.7	-	10	
70	GT22F-N27	GT22F-N27L	GT22F-N27LE	22Fr	7.3	-	10	
71	GT24F-N27	GT24F-N27L	GT24F-N27LE	24Fr	8.0	-	10	
72	GT26F-N27	GT26F-N27L	GT26F-N27LE	26Fr	8.7	-	10	
73	GT12F-N30	GT12F-N30L	GT12F-N30LE	12Fr	4.0	-	5	30
74	GT14F-N30	GT14F-N30L	GT14F-N30LE	14Fr	4.7	-	5	
75	GT16F-N30	GT16F-N30L	GT16F-N30LE	16Fr	5.3	-	10	
76	GT18F-N30	GT18F-N30L	GT18F-N30LE	18Fr	6.0	-	10	
77	GT20F-N30	GT20F-N30L	GT20F-N30LE	20Fr	6.7	-	10	
78	GT22F-N30	GT22F-N30L	GT22F-N30LE	22Fr	7.3	-	10	
79	GT24F-N30	GT24F-N30L	GT24F-N30LE	24Fr	8.0	-	10	
80	GT26F-N30	GT26F-N30L	GT26F-N30LE	26Fr	8.7	-	10	
81	GT12F-N35	GT12F-N35L	GT12F-N35LE	12Fr	4.0	-	5	35
82	GT14F-N35	GT14F-N35L	GT14F-N35LE	14Fr	4.7	-	5	
83	GT16F-N35	GT16F-N35L	GT16F-N35LE	16Fr	5.3	-	10	
84	GT18F-N35	GT18F-N35L	GT18F-N35LE	18Fr	6.0	-	10	
85	GT20F-N35	GT20F-N35L	GT20F-N35LE	20Fr	6.7	-	10	

86	GT22F-N35	GT22F-N35L	GT22F-N35LE	22Fr	7.3	-	10	
87	GT24F-N35	GT24F-N35L	GT24F-N35LE	24Fr	8.0	-	10	
88	GT26F-N35	GT26F-N35L	GT26F-N35LE	26Fr	8.7	-	10	
89	GT12F-N40	GT12F-N40L	GT12F-N40LE	12Fr	4.0	-	5	40
90	GT14F-N40	GT14F-N40L	GT14F-N40LE	14Fr	4.7	-	5	
91	GT16F-N40	GT16F-N40L	GT16F-N40LE	16Fr	5.3	-	10	
92	GT18F-N40	GT18F-N40L	GT18F-N40LE	18Fr	6.0	-	10	
93	GT20F-N40	GT20F-N40L	GT20F-N40LE	20Fr	6.7	-	10	
94	GT22F-N40	GT22F-N40L	GT22F-N40LE	22Fr	7.3	-	10	
95	GT24F-N40	GT24F-N40L	GT24F-N40LE	24Fr	8.0	-	10	
96	GT26F-N40	GT26F-N40L	GT26F-N40LE	26Fr	8.7	-	10	
97	GT12F-N45	GT12F-N45L	GT12F-N45LE	12Fr	4.0	-	5	45
98	GT14F-N45	GT14F-N45L	GT14F-N45LE	14Fr	4.7	-	5	
99	GT16F-N45	GT16F-N45L	GT16F-N45LE	16Fr	5.3	-	10	
100	GT18F-N45	GT18F-N45L	GT18F-N45LE	18Fr	6.0	-	10	
101	GT20F-N45	GT20F-N45L	GT20F-N45LE	20Fr	6.7	-	10	
102	GT22F-N45	GT22F-N45L	GT22F-N45LE	22Fr	7.3	-	10	
103	GT24F-N45	GT24F-N45L	GT24F-N45LE	24Fr	8.0	-	10	
104	GT26F-N45	GT26F-N45L	GT26F-N45LE	26Fr	8.7	-	10	
105	GT12F-N50	GT12F-N50L	GT12F-N50LE	12Fr	4.0	-	5	50
106	GT14F-N50	GT14F-N50L	GT14F-N50LE	14Fr	4.7	-	5	
107	GT16F-N50	GT16F-N50L	GT16F-N50LE	16Fr	5.3	-	10	
108	GT18F-N50	GT18F-N50L	GT18F-N50LE	18Fr	6.0	-	10	
109	GT20F-N50	GT20F-N50L	GT20F-N50LE	20Fr	6.7	-	10	
110	GT22F-N50	GT22F-N50L	GT22F-N50LE	22Fr	7.3	-	10	
111	GT24F-N50	GT24F-N50L	GT24F-N50LE	24Fr	8.0	-	10	
112	GT26F-N50	GT26F-N50L	GT26F-N50LE	26Fr	8.7	-	10	
113	GT12F-N55	GT12F-N55L	GT12F-N55LE	12Fr	4.0	-	5	55
114	GT14F-N55	GT14F-N55L	GT14F-N55LE	14Fr	4.7	-	5	
115	GT16F-N55	GT16F-N55L	GT16F-N55LE	16Fr	5.3	-	10	

116	GT18F-N55	GT18F-N55L	GT18F-N55LE	18Fr	6.0	-	10	
117	GT20F-N55	GT20F-N55L	GT20F-N55LE	20Fr	6.7	-	10	
120	GT22F-N55	GT22F-N55L	GT22F-N55LE	22Fr	7.3	-	10	
119	GT24F-N55	GT24F-N55L	GT24F-N55LE	24Fr	8.0	-	10	
120	GT26F-N55	GT26F-N55L	GT26F-N55LE	26Fr	8.7	-	10	
121	GT12F-N60	GT12F-N60L	GT12F-N60LE	12Fr	4.0	-	5	60
122	GT14F-N60	GT14F-N60L	GT14F-N60LE	14Fr	4.7	-	5	
123	GT16F-N60	GT16F-N60L	GT16F-N60LE	16Fr	5.3	-	10	
124	GT18F-N60	GT18F-N60L	GT18F-N60LE	18Fr	6.0	-	10	
125	GT20F-N60	GT20F-N60L	GT20F-N60LE	20Fr	6.7	-	10	
126	GT22F-N60	GT22F-N60L	GT22F-N60LE	22Fr	7.3	-	10	
127	GT24F-N60	GT24F-N60L	GT24F-N60LE	24Fr	8.0	-	10	
128	GT26F-N60	GT26F-N60L	GT26F-N60LE	26Fr	8.7	-	10	
129	GT12F-N65	GT12F-N65L	GT12F-N65LE	12Fr	4.0	-	5	65
130	GT14F-N65	GT14F-N65L	GT14F-N65LE	14Fr	4.7	-	5	
131	GT16F-N65	GT16F-N65L	GT16F-N65LE	16Fr	5.3	-	10	
132	GT18F-N65	GT18F-N65L	GT18F-N65LE	18Fr	6.0	-	10	
133	GT20F-N65	GT20F-N65L	GT20F-N65LE	20Fr	6.7	-	10	
134	GT22F-N65	GT22F-N65L	GT22F-N65LE	22Fr	7.3	-	10	
135	GT24F-N65	GT24F-N65L	GT24F-N65LE	24Fr	8.0	-	10	
136	GT26F-N65	GT26F-N65L	GT26F-N65LE	26Fr	8.7	-	10	

Table 2 Model and Component of Silicone Gastrostomy Tube

Model	Component
GT Series Y-type	Y-type, YC-type, Y2-type, YC2-type, E-type, EC-type, E2-type, EC2-type include Tube body, Balloon, Feeding joint, Bent fixed ring/Fixed ring, Filling joint
GT Series E-type	
	YL-type, YCL-type, YL2-type, YCL2-type, EL-type, ECL-type, EL2-type, ECL2-type include Tube body, Balloon,

	Feeding joint, Bent fixed ring/Fixed ring, Filling joint, Guide bar
GT Series N-type	N-type include Tube body, Balloon, Feeding joint, Filling joint;NL-type include Tube body, Balloon, Feeding joint, Filling joint, Feeder, Injector, Gauze sponges, Guide bar, Straight feeding tube, Bent feeding tube;NLE-type include Feeding E joint Feeder; E-type Straight feeding tube, E-type Bent feeding tube, Injector, Gauze sponges, Tube body, Balloon, Feeding joint, Filling joint, Guide bar
VED Series	Tube body, Balloon, Feeding joint, Fixed ring, Filling joint